

Epaderm[®] Cream

IMPROVED SKIN HYDRATION IN 2 WEEKS

DOCTOR[®] RECOMMENDED

- ✓ Dry Skin
- ✓ Eczema
- ✓ Psoriasis
- ✓ Fragrance & SLS¹ free
- ✓ Suitable for all ages, including babies

50g e

Mölnlycke

Epaderm Cream - recommended by doctors for the management of dry skin conditions, eczema and psoriasis.

Epaderm Cream is an emollient formulated with clinically proven ingredients that moisturise and soften the skin. Emollients are especially effective if applied immediately after washing or swimming. More at www.epaderm.com

Mölnlycke Health Care AB, Gamlestadvägen 3C, Box 13080, SE-402 52 Göteborg, Sweden www.molnlycke.com Made in UK

REF 99400821
427073-07

25°C

CE 2797









Fire hazard. Do not smoke or go near naked flames; clothing & bedding with this product dried on them can catch fire easily .

Component no: 427073-07

Dimensions: 120mm x 55mm (2 ply label)

Created by: Natalie Whiteside

Colour information

-  PMS 361
-  PMS Cool Grey 10
-  PMS 2577
-  Varnish free 68mm x 20mm
-  Diecut
-  PMS 152
-  CMYK Yellow
-  Black

Please use PD-589365 and PD-589893 when reviewing this artwork

Instructions for use: Always wash your hands thoroughly before applying Epaderm Cream. Depress pump head repeatedly until cream is dispensed.

Can be used as follows:

Emollient: Apply liberally to the affected area and massage well into the skin. Use as often as required or as directed by your health care professional.

Skin cleanser: Use as required when washing or in the shower.

Precautions: Do not use Epaderm Cream on patient and/or user with known hypersensitivity to the ingoing materials/components of the product. If your condition worsens or you experience any undesirable effects, stop using Epaderm Cream and discuss with your health care professional. For external use only. Avoid contact with the eyes. Do not use after expiry date.

Warning: Do not smoke or go near naked flames – risk of severe burns. Fabric (clothing, bedding, dressings etc) that has been in contact with this product burns more easily and is a serious fire hazard. Washing clothing and bedding may reduce product build-up but not totally remove it.

Ingredients: Yellow Soft Paraffin BP, Liquid Paraffin Ph. Eur., Cetomacrogol Emulsifying Wax BP, (contains Cetostearyl Alcohol and Macrogol Cetostearyl Ether 22), Chlorocresol, Glycerine BP, Purified Water EP.

Storage: Store below 25°C in a dry place. Keep out of the reach of children. Disposal should be handled according to local environmental procedures.

Other information: If any serious incident has occurred in relation to the use of Epaderm Cream, it should be reported to Mölnlycke Health Care. The Mölnlycke and Epaderm trademark, name and logo are registered globally to one or more of the Mölnlycke Health Care Group of Companies.

¹SLS is an abbreviation for Sodium Lauryl Sulphate

²Source UK Prescription Cost Analysis Data 2018-2019